





Ultra Thin

Skin Doctor

Hydrocolloid Circular Band

Choose depending on your scar size!

✓ What is 'Skin Doctor' ?

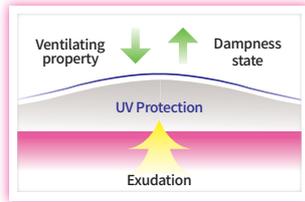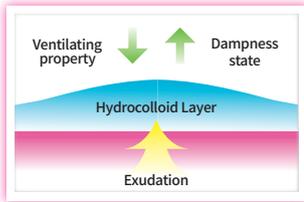

- Thin, Soft and Pliable
- Highly absorptive performance
- Waterproof and blocking effect against external invading bacteria


Mixed 46 Sheets

12mm x 15 Sheets


10mm x 24 Sheets

8mm x 7 Sheets

SIZE

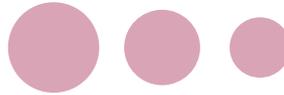

✓ How it works ?

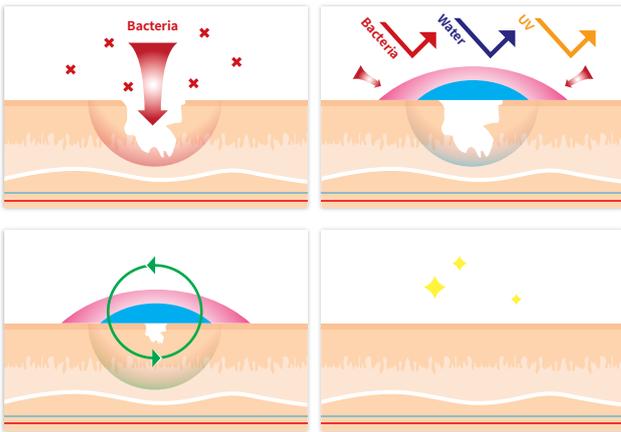

1 2
3 4

Hydrocolloid Structure

Ventilating PU Film layer : PU Film layer has ventilating property and keeps a fixed dampness state. PU Film layer also blocks contamination, water and bacteria from the outside.

Hydrocolloid layer : It absorbs exudation from the injury and keeps a fixed dampness state so scar or scab is not occurred.

How to use

- 1) Wash the injured region with water or saline solution and make it dry.
- 2) Choose bigger size than the injury and stick a band then press well.
- 3) Once the injured region swells wide owing to exudation as time passes, exchange it.
- 4) It can be used for 1-2 days depending on the injured extent and personal skin.



Skin Doctor

E info@ids.co.kr
Tel 82-2-2025-1151~3
Fax 82-2-2025-1154
www.ids.co.kr

ids
Innovation for better life